

derma+flex[®]

GEL TISSUE ADHESIVE™

Fast & Effective

THE PRODUCT

The patented Derma+Flex medical adhesive is formulated as a controlled flow gel.

Derma+Flex is a flexible gel adhesive providing a supple, secure and transparent protective microbial barrier over most incisions, minor wounds and abrasions.

Derma+Flex is sterile, viscous and easy to apply. It stays where you want without running. Single layer applications and faster set-times are facilitated by high viscosity. There is usually no need to apply multiple layers.

Derma+Flex is solvent-free and will not sting when correctly applied. Its unique proprietary formulation and method of setting means there is negligible exothermic reaction (heat generation). It naturally sloughs off without causing trauma.

Derma+Flex is ideally suited for areas of skin exposed to flexing and moisture, including between fingers and toes, fingertips, chins, eyebrows, elbows and knees, which are difficult to seal with traditional bandages.

Derma+Flex obtains optimum success if stored at room temperature (below 30°C) away from moisture and heat.

For more information or to order please contact: marketing@chemence.com



derma+flex

GEL TISSUE ADHESIVE™

Fast & Effective

FREQUENTLY ASKED QUESTIONS

Does Derma+Flex sting?

No, under normal conditions it will not sting, which is caused by solvents or heat producing a burning sensation.

Derma+Flex contains no solvents and has a unique formula and curing mechanism generating little or no heat.

How many times can you use the adhesive?

The sterile product is intended for single use.

Is Derma+Flex waterproof?

Yes.

How long is the shelf life of Derma+Flex?

Shelf life is 24 months at recommended conditions.

Recommended storage conditions: below 30°C (86°F) away from moisture and direct heat.

Does Derma+Flex provide a microbial barrier?

Yes.


CHEMENCE MEDICAL PRODUCTS, INC.

Chemence Medical Products, Inc. (CMPI) has been an innovative pioneer in cyanoacrylate technology for medical use. Emphasis has been placed upon research and development to promote new and improved products with high performance and beneficial technology characteristics. CMPI products are protected by a portfolio of patents covering various aspects of cyanoacrylate technology for medical use, including sterilization, packaging, viscosity and bio-absorbable formulations. CMPI has become a world leader in medical cyanoacrylate devices and products and is committed to be at the forefront of technology in this field.

European Distributor



Product: Derma+Flex Gel Tissue Adhesive is used to close wounds and incisions. Derma+Flex gel tissue adhesive gives a flexible waterproof bandage, covering and protecting the area providing an antimicrobial barrier.

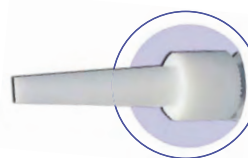

DERMA+FLEX FINE APPLICATOR

Allows fine applications where looks (cosmetic issues) are considered



DERMA+FLEX DAUBER TIP

Allows a flow controlled application to the surface, that provides a flexible waterproof bandage



Before Derma+Flex



After Derma+Flex



NHS Code
FVF010